

Diagnostic Scoring Guide

English

Data signals dashboard



Start/End of evaluation—displayed at the beginning or end of the analysis period from which statistics are calculated.

Artifact data are those that has been excluded from the analysis.

Events—Hypopnoea, Central, Obstructive, Mixed, and Unclassified apnoeas are all as per AASM parameters that have been selected for analysis. Events can be edited or deleted (including Desaturation).

To navigate:

1. Click the left or right arrow on the hypnogram to move the display to a 3-minute epoch.
2. Press CTRL + left or right arrow key on the keyboard to move the display to a 90-second increment.
3. Click and drag the view bar within the hypnogram to display any time period.

Note: Data signal types and their positions depend on the HST device. For more information on signal types, refer to the device manual.




Step-by-step manual scoring


Data is auto-scored when data is downloaded into AirView. If required, you can manually score the raw data signals which can override the auto-scoring.

To manually score:


1. Select **Raw data signals** from the Diagnostic tab of a patient's file.
2. Manually change data as required:

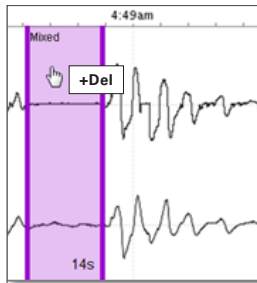

To delete, hover over the event and press **Delete** on the keyboard.

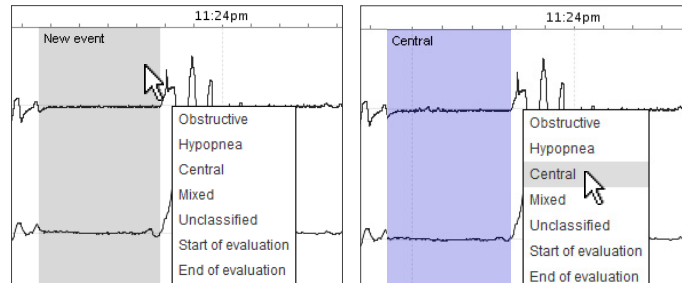

To insert, click the left mouse button at the start of the event and drag the cursor to the end of the event. Select the event type to finish.

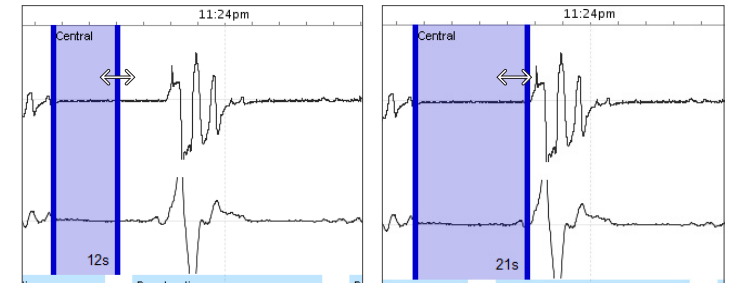

To change an event duration, move the cursor to the beginning or end of an event and drag the left-right arrow to the desired position.

3. Click **Save**. Statistics on the diagnostic screen are automatically updated.

To reset to auto-score:

1. From the **Statistics** panel under the **Diagnostic** tab, select the AASM analysis guideline (if applicable) from the drop down menu.
2. Click **Reanalyse**.